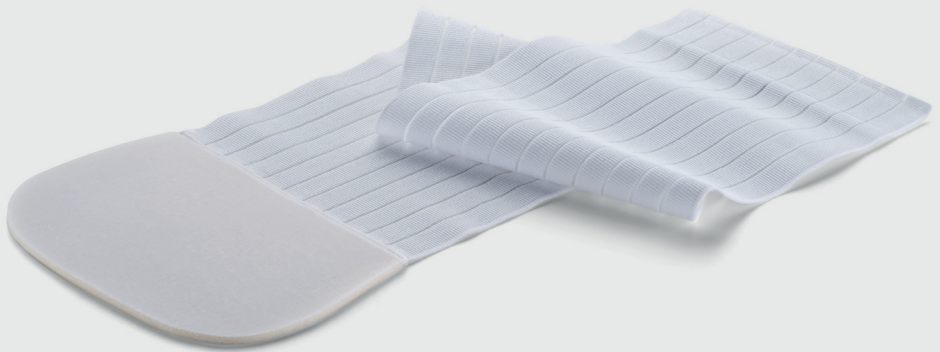
biber

Entlastende Stützbandage

Anleitung
Instructions
Instructions

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Scope of delivery:

1 x Entlastende Stützbandage

Read the instructions for use completely and carefully!

Purpose

To activate and support the muscles of the abdomen and back through external pressure (compression). To support the spine and promote an upright posture. The support bandage provides support and stability after an operation to relieve the wound.

- slipped discs
- sciatica pain
- lumbago
- pain from overstraining
- rehabilitation after operations

For personal use.

Contraindications / Unsuitability

Wearing the product is only recommended for the following clinical conditions following consultation with your doctor:

- skin diseases/injuries in the affected part of the body, especially inflammatory conditions such as swelling, redness and overheating
- allergic reactions to the material
- sensory disturbances and movement disorders, e.g. with diabetes
- open wounds or internal injuries
- in case of infants or persons who would not notice if the product is placed uncomfortably

Instructions for safe use of the product

There are no known side effects that involve the entire organism. Proper use is a prerequisite. Product liability is excluded in the case of improper use.

- If the posture and support brace is too tight or feels uncomfortable, remove it immediately!
- Wear the posture and support brace only for an hour at a time, not permanently.
- Consult your doctor immediately if the symptoms worsen or you notice any changes in your condition.
- All serious incidents involving the product must be reported to the manufacturer and the responsible authorities of the Member State where the user and/or patient is located.

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Federal Ministry of Health
 Dept. Pharmaceuticals and Medical Devices
 Radetzkystrasse 2, 1030 Wien/Vienna
 E-Mail: meddev@bmgf.gv.at

Für Schweiz

Swissmedic, Swiss Agency for Therapeutic Products
 Medical Devices Division
 Hallerstrasse 7, 3012 Bern, Switzerland

Do not use

- on small children or people who cannot call for attention if the posture and support brace is uncomfortably tight.
- while sleeping.

The comfortable Velcro fastener closes the posture and support brace when it is put on. It is not a fault if, when opening the Velcro fastener, small fibres pull out of the surface material of the harness.

Use

Fig. 1: Hold the support bandage with the Velcro fastener (inside) in your right hand and pass it around your back. For optimal support, pull the bandage away from the body on both sides.


Fig. 2 + 3: Close the front with the Velcro fastener on the corresponding counterpart.

Cleaning

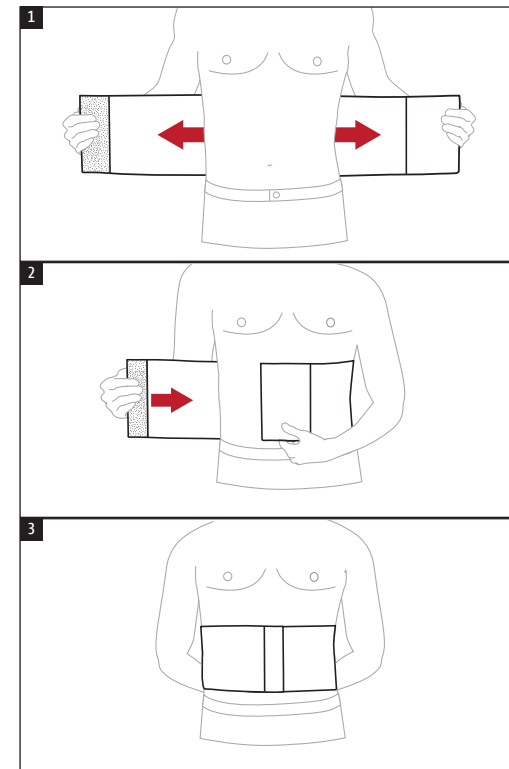The brace can be washed with the household laundry – see the care instructions. Close the Velcro fasteners before washing to prevent damage to other items of laundry.

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Biber Umweltprodukte Versand GmbH
Hydas GmbH & Co. KG, Hirzenhainer Straße 3,