
biber

Seniorenstuhl

Anleitung
Instructions
Istruzioni

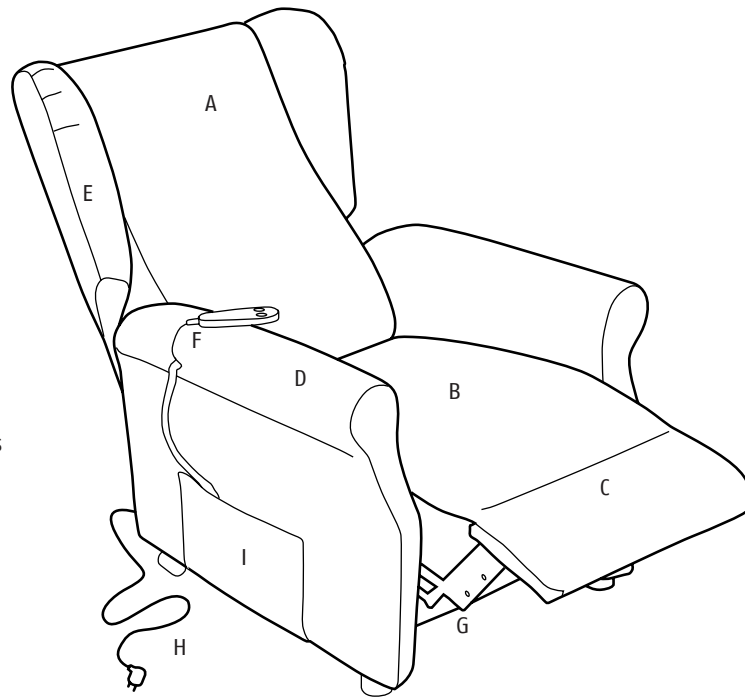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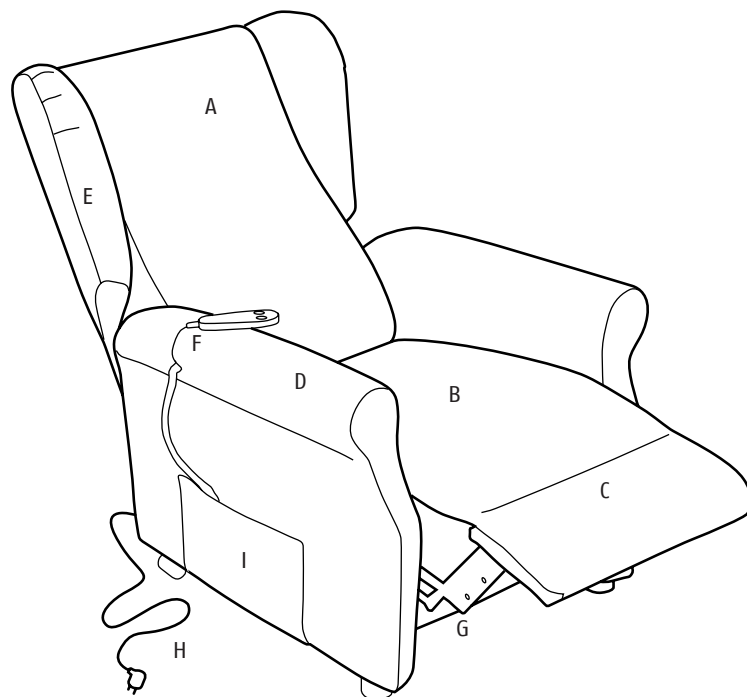
- A Backrest
- B Seat
- C Legrest
- D Armrest
- E Wing Backrest
- F Control Handset
- G Mechanism
- H Power Cord
- I Storage Pocket

Warning!

Instructions and cautions for safe use. Before using this device, please read the instructions carefully, particularly the notes, warnings and safety instructions that follow them. Please keep this manual and the relevant guide for future reference. Should you transfer this product to a third party, please also give them this complete manual.

Please note

Should some of the information contained in this manual be difficult to understand or if you have any questions, please contact the manufacturer at the address shown on the last page before using the chair.

Warning

After unpacking the chair, check its integrity and whether there is any transport damage. In case of doubt, do not use the chair and contact our assistance service.

Warning

Remove any communication materials such as labels, tags, information booklet etc. from the armchair before using it.

Warning

Risk of choking or suffocation. Children should not play with the packaging. Keep the plastic bag out of reach of children.

Dear customer, we kindly ask you to pay attention to our safety instructions. It is very important that you read and understand the symbols used. The meaning of the symbols used is explained below.

Warning/danger symbol

Indicates any situation that is hazardous to objects or people. Take maximum care and follow the instructions carefully.

Electrical danger

Indicates any electrical situation that is hazardous to people. Take maximum care and follow the instructions carefully.

Prohibition symbol

Indicates improper use of the chair that could result in serious injury or serious danger. Take maximum care and follow the instructions carefully.


Precautions for use

This armchair must be used only for the purpose for which it was designed, as an electric armchair for domestic use. Any other use is considered not to be in accordance with the intended use and therefore dangerous.

This armchair is designed to be used by people weighing no more than 110 kg. This armchair is not suitable for use by more than one person at the same time.

Before connecting the armchair to the power supply, check that the voltage and frequency shown in the technical data of this armchair correspond to those of the available power supply. Technical data can be found on the armchair and on the adapter, if provided.

The armchair can be used by children of at least 8 years of age and by persons with reduced physical, sensory or mental abilities, subject to continuous supervision or after they have received instruction on the safe use of the armchair and understand the potential hazards involved. Children must not be allowed to play with the armchair.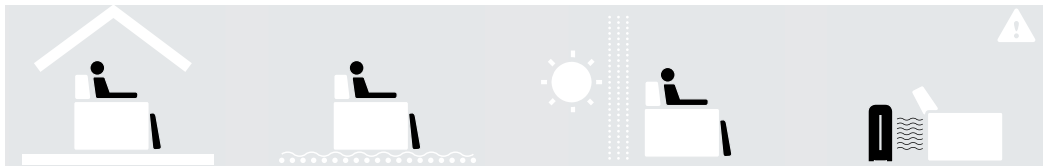

Use the armchair only in domestic settings on a flat, level and stable surface.

If used on a carpeted floor, the use of an additional rug is recommended to prevent any damage to the carpet caused by the weight of the armchair.

Place in an area not exposed to direct sunlight, in order to avoid changes to the upholstery fabric of the armchair.

Avoid placing the armchair next to any heat source (such as ovens, radiators, etc.).


Do not use in environments with high humidity and avoid contact with water, liquids and sprays.

Check that there are no persons, animals or objects within the range of action of the backrest.

Check that there are no people, animals or objects within the range of action of the legrest.

Place the armchair at a distance of more than 70 cm from the wall.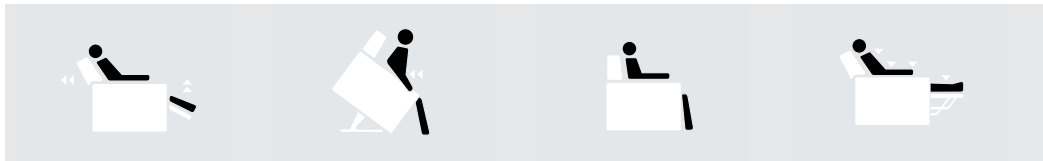

Raise the legrest first and then recline the backrest.

Sit down carefully when the chair is raised.

Keep your legs relaxed while raising the legrest.

Distribute your weight on the armchair as evenly as possible.

Unlock the roller system only when the armchair is unoccupied.

Unlock the roller system only after moving the chair, before using it.

Do not recline the backrest when the chair is raised.

Do not bend forwards with the legrest raised.

Do not sit, climb or jump on the backrest or armrests.

Do not touch the metal parts when moving, risk of injury.

Do not apply any weight or leverage on the raised legrest.

Lower the legrest and return the backrest to the upright position before getting up from the armchair.

Do not put your feet down to the ground during the movement.

Do not use the armchair if there is any visible damage.

Do not obstruct the armchair in any way.

Do not stand up by applying weight to only one side of the armchair.

Do not throw yourself into the armchair.

Electrical precautions


Plug the power cord into the socket. Risk of short circuit or electrical fire.

Whenever the armchair is not in use, make sure the power cord is unplugged and the remote control is placed in the storage pocket.

Do not pull on the power cord or the armchair in order to unplug the power cord from the power socket.

Unplug the power cord before cleaning and maintenance.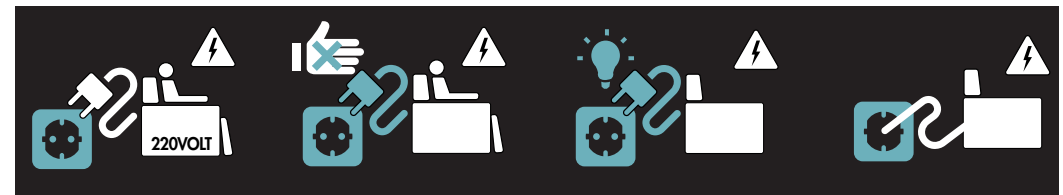


Do not use with a voltage different from the one specified on the label. Risk of short circuit or fire.

Do not tamper with the power cord, or the power socket. Risk of shock, short circuit and fire.

In the event of a power failure, turn off the product and remove the plug from the power socket. Risk of accidents.

Do not leave the product connected to the power supply when unattended. Risk of fire, shock, short circuit.

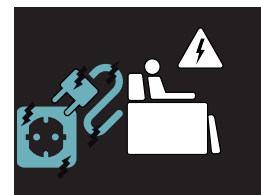Do not use if the power cord, plug or power socket is damaged. Risk of shock, short circuit and fire.

Do not use wearing wet clothing. Do not use if the armchair is wet. Do not use with wet hands or with wet or naked feet. Risk of injuries or shock.

Cleaning and maintenance

We recommend that you read the instructions below carefully and avoid any operation which is not described there.

For routine maintenance of the armchair, it is advisable to dust the external cover of the armchair and the control handset with a dry cloth, avoiding the use of chemical detergents.

Warning

- Always unplug the power cord from the power supply before cleaning or maintenance.
- Ensure the integrity of the power cord and periodically check the full functionality of the electric motors.
- The use of liquids and/or flammable substances is prohibited for cleaning electrical parts.
- Before using the armchair again, make sure all cleaned parts are completely dry.
- If the armchair is not used for an extended period of time, store it somewhere with low humidity and dust, in order to avoid mould on the cover or rust on the frame mechanism.

We recommend dusting the product with a soft brush and cleaning it gently with a vacuum cleaner from time to time.


For water-based stains, such as coffee, tea or soft drinks, it is advisable to pat the armchair lightly with a cloth to absorb the liquid; do not rub.

Stains caused by greasy substances such as oil, ketchup, mayonnaise or chocolate, should be gently removed from the surface using a paper towel, paying attention not to press too hard, so as to prevent dirt from entering the pores of the leather. With a clean cloth soaked in a very diluted neutral soap solution, gently rub from the outside towards the inside of the stain. Dry immediately with a clean cloth. Use only water and neutral, colourless soap.

Before using any detergent, it is advisable to test its effects on a small hidden area of the armchair, in order to avoid damaging the armchair.

- Do not use the armchair if any visible damage is present.
- Insert the plug into the power socket.
- The user must be seated in the correct position before operating the armchair.

Functions and control handset

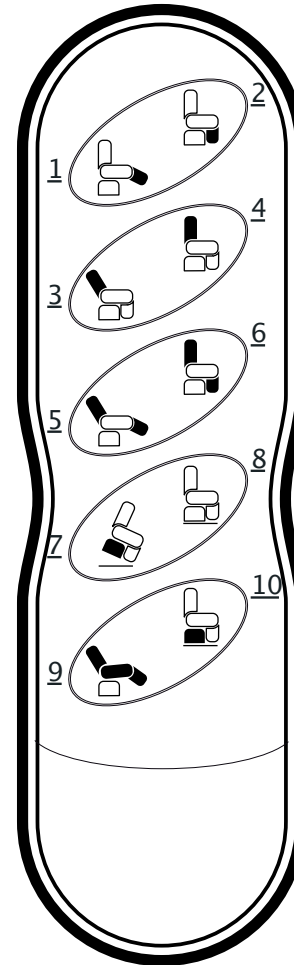Sitting position menu

Pressing button 1 raises the legrest. The legrest can be stopped in the desired position by releasing the button. Pressing button 2 lowers the legrest. The legrest can be stopped in the desired position by releasing the button. Pressing button 3 reclines the backrest. The backrest can be stopped in the desired position by releasing the button. Pressing button 4 raises the backrest. The backrest can be stopped in the desired position by releasing the button. Pressing button 5 reclines the backrest and raises the legrest. The backrest and legrest can be stopped in the desired position by releasing the button. Pressing button 6 raises the backrest and lowers the legrest. The backrest and legrest can be stopped in the desired position by releasing the button.

Standing up/Lift function

Pressing button 7 raises the chair at the back. The chair can be stopped in the desired position by releasing the button. To lower the chair again, press and hold button 8.

Front tilting function

Pressing button 9 raises the chair at the front. The chair can be stopped in the desired position by releasing the button. To lower the chair again, press and hold button 10.

Lifting function parallel to the ground

By activating the front tilting function together with the lift function, you can lift the chair parallel to the ground by about 20 cm. Then press button 9 to raise the chair at the front. Next, press button 7 to raise the chair at the back. Release button 7 when the chair is parallel to the ground. To lower the chair to the ground, first press button 8 to lower the back and then button 10 to lower the front.

Problems and solutions

This section contains the most common problems related to the use of the armchair. If you are unable to get your armchair operating with the following information, please contact our Service Centre.

Problem	Solution
The armchair does not turn on.	Check that the power plug is properly plugged into the socket.
The armchair does not work.	Check that the position of the roller system lever located at the bottom back of the seat is not locked (Roller system unlocked).
The limit switch of any of the movements is not reached.	Check that the armchair is not too close to a wall which the movements is not reached. Make sure that there are no objects stuck in the mechanism of the armchair.

Disposal

The packaging of the armchair is made from recyclable materials. Dispose of it in accordance with environmental protection standards.

According to Art. 26 of the Legislative Decree of 14 March 2014, no. 49 "Implementation of Directive 2012/19/EU concerning electrical and electronic equipment waste (WEEE)", the rubbish bin symbol on the equipment or on its packaging indicates that the product must be collected separately at the end of its life cycle. The user must take the equipment to the appropriate waste centre. Alternatively, it is possible to return the equipment you want to dispose of to the dealer when you purchase new equipment of a similar type. In the case of electronic product retailers with a sales area of at least 400 square metres, it is also possible to return any electronic devices to be disposed of with dimensions of less than 25 cm free of charge and without any obligation to make a new purchase. Sending the disposed equipment for recycling helps to avoid any negative effects on the environment and health and promotes the recycling of all the materials used in the equipment.

Warranty

The warranty on this armchair is valid for 24 months and is effective from the delivery date. The warranty is limited to manufacturing defects and does not cover any liability for direct or indirect damage.


Any damage caused during transportation carried out by a third party, by incorrect installation or maintenance, negligence or unauthorised modifications will not be covered by the warranty.

Electrical components and normal wear and tear are also excluded from the warranty.

Terms

- The product is tested and covered by warranty for 24 months.
- The warranty starts on the invoice date, unless the purchaser can prove that the delivery date was later than the invoice date.
- The product is guaranteed against material and/or manufacturing defects, the warranty consists in the replacement or repair of the defective parts.
- Defects resulting from the following cases are excluded from the warranty: improper or incorrect use of the product, use which does not conform with the instructions in the instruction booklet, damage caused during transportation carried out by third parties, incorrect maintenance, unauthorised modifications, damage due to a voltage connection other than that provided for the appliance or different from the limit established by the IEC standards. Also excluded from any form of warranty are all parts subject to wear and tear. Any liability for direct or indirect damage caused by the use of the product is also excluded.
- Home repairs must be agreed in advance with Biber. The warranty will be rendered null and void if the product has been tampered with or repaired by unauthorised personnel.
- Repairs can only be carried out by personnel authorised by Biber.

Massage function – only for the chair with massage function

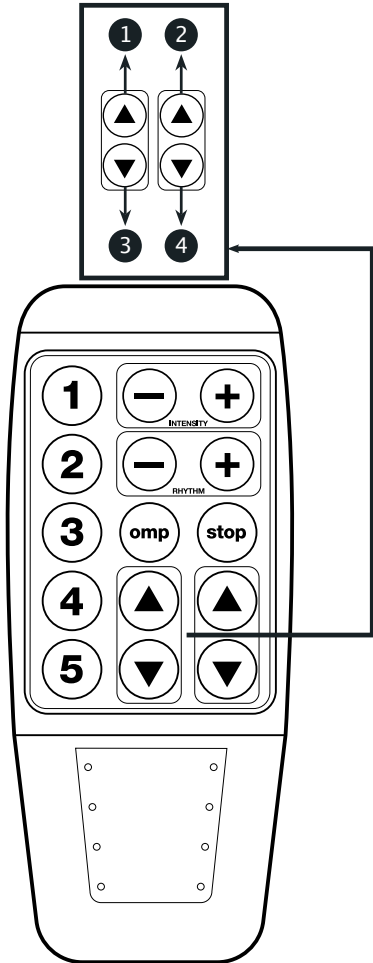Precautions for massage versions

- The following types of user should consult their doctor before using the product.
- People being treated for medical treatment, with infection or injury. Users with problems or deformities in their bone structure. Pregnant women.
- People recovering from surgery.
- People with heart disorders and with pace-makers.
- People with mental and/or physical disabilities must always be supervised when using the product.
- People with severe fever.

If the user feels unwell during use or suffers pain due to excessive stimulation, turn off the armchair immediately and stop the session.

Sitting menu

Pressing button 4 reclines the backrest. The backrest can be stopped in the desired position by releasing the button.
 Pressing button 3 raises the legrest. The legrest can be stopped in the desired position by releasing the button.
 Pressing button 2 raises the backrest. The backrest can be stopped in the desired position by releasing the button.
 Pressing button 1 lowers the legrest. The legrest can be stopped in the desired position by releasing the button.

Lift function

Before raising the chair to the Lift position, make sure that the backrest is completely in the upright position by pressing button 2. Pressing button 1 raises the chair. The chair can be stopped in the desired position by releasing the button. To lower the chair again, press and hold button 3.

Activating and stopping the vibrating massage feature

Pressing the button once activates the vibrating massage.
 You can activate 5 vibrating areas:
 1 High back
 2 Low back
 3 Back seat
 4 Front seat
 5 Legrest
 Pressing the button switches of the vibrating massage.

+ –

Adjusting the massage intensity

(5 intensity levels)
 Pressing the + button increases the intensity one level at a time. Pressing the – button decreases the intensity one level at a time.

Massage programs

(3 programs)
 Activate the programs by pushing the omp button
 Pressing the omp button once activates the Tappingmassage program. One vibrating device at a time will operate from High back to Legrest.
 Pressing the omp button twice activates the Ondamassage program. All vibrating devices will operate at speed.
 Pressing the omp button a third time activates the Flatmassage program. All vibrating devices will operate simultaneously.

+ –


Adjusting the massage speed

(12 speed levels)
 Pressing the + button increases the speed one level at a time. Pressing the – button decreases the speed one level at a time.

Problems and solutions

This section contains the most common problems related to the use of the armchair. If you are unable to get your armchair operating with the following information, please contact our Service Centre.

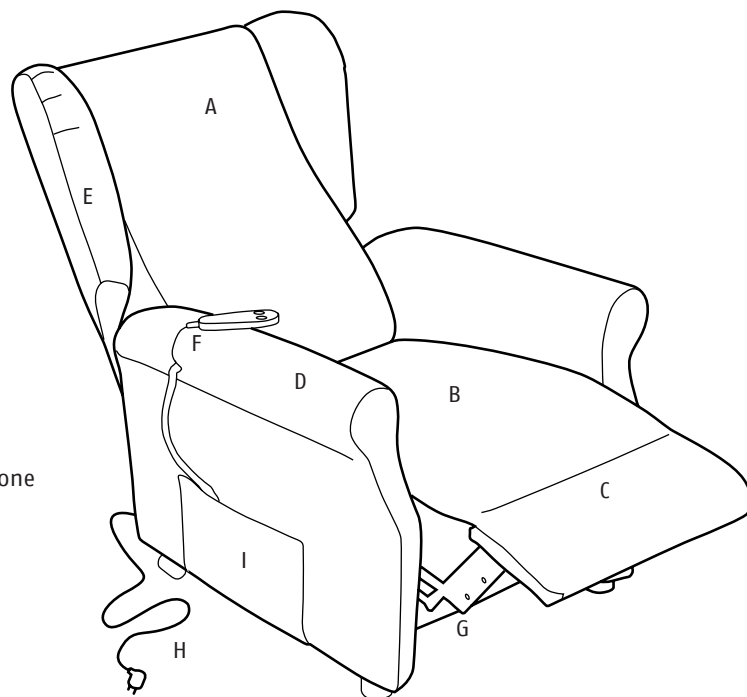Problem	Solution
The vibrating massage function stops.	The vibration system pauses for 5 minutes every 12 stops.

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