**biber**

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Anleitung  
Instructions  
Instructions

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Version: 2537Gba\_202003Sz

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**Posture and support brace 458453**

Version: 2537Gba\_202003Sz

**Scope of delivery:****1 x Posture and support brace**

Read the instructions for use completely and carefully!

**Purpose**

The posture and support brace is designed to activate the abdominal and back muscles through external pressure (compression) and at the same time to pull the shoulders back with the shoulder straps. The harness straightens the upper body and in doing so relieves the lower back. The stabilisation and relief of the lower back and surrounding musculature is relaxing and thus relieves pain.

Supports the stomach and back, and promotes an upright posture in the case of

- slipped discs
- pain from overstraining
- poor posture

Can be worn for an hour at a time by private users.

**Contraindications / Unsuitability**

In the case of the following pathologies, the posture and support brace should only be used after consultation with your doctor:

- skin diseases/injuries in the affected part of the body, especially inflammatory conditions such as swelling, redness and overheating

- sensory disturbances and movement disorders, e.g. with diabetes
- open wounds or internal injuries
- allergic reactions to the material of the posture and support brace

**Instructions for safe use of the product**

There are no known side effects that involve the entire organism. Proper use is a prerequisite. Product liability is excluded in the case of improper use.

- If the posture and support brace is too tight or feels uncomfortable, remove it immediately!
- Wear the posture and support brace only for an hour at a time, not permanently.
- Consult your doctor immediately if the symptoms worsen or you notice any changes in your condition.
- All serious incidents involving the product must be reported to the manufacturer and the responsible authorities of the Member State where the user and/or patient is located.

**For Germany**

Federal Institute for Drugs and Medical Devices  
Medical Devices Division  
Kurt-Georg-Kiesinger-Allee 3  
53175 Bonn  
<https://www2.bfarm.de/medprod/mpsv/>

**For Austria**

Federal Ministry of Health  
Dept. Pharmaceuticals and Medical Devices  
Radetzkystrasse 2, 1030 Vienna  
Email: [meddev@bmgf.gv.at](mailto:meddev@bmgf.gv.at)

**For Switzerland**

Swissmedic, Swiss Agency for  
Therapeutic Products  
Medical Devices Division  
Hallerstrasse 7, 3012 Bern, Switzerland

**Do not use**

- on small children or people who cannot call for attention if the posture and support brace is uncomfortably tight.
- while sleeping.

The comfortable Velcro fastener closes the posture and support brace when it is put on. It is not a fault if, when opening the Velcro fastener, small fibres pull out of the surface material of the harness.

**Use / Putting on the support brace:**

Fig. 1: Hold the brace with the Velcro fastener (inside) in your right hand and pass it around your back. First fold in the left side, then the right. The first Velcro fastener makes it easier to put on. Then use light pressure to attach the second Velcro fastener to any point on the harness.



Fig. 2: Pass the two shoulder straps forwards over the shoulder and then backwards under the armpits. Attach the straps to any point on the harness. The firmer you pull, the greater the supporting force on the shoulders.

Warning! Do not fasten the shoulder straps together, as this could damage the material.

**Cleaning**

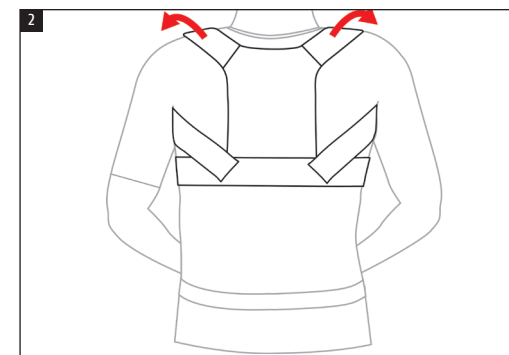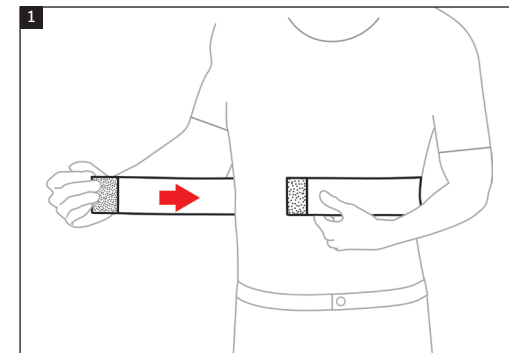The brace can be washed with the household laundry – see the care instructions. Close the Velcro fasteners before washing to prevent damage to other items of laundry.

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Biber Umweltprodukte Versand GmbH
Hydax GmbH & Co. KG, Witzenhainer Straße 3,