biber

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Read the instructions for use completely and carefully!

Purpose

The bandage supports and stabilises. Targeted pressure applied to tendons and muscles can reduce pain and thereby relieve the muscular structure. The objective is to reduce the intake of painkillers and increase the mobility of the patient.

Supports the musculature in case of

- joint complaints, ligament and muscle weakness
- stretching, spraining, bruising
- tendonitis
- pain from overstraining
- rehabilitation after operations

For personal use.

Contraindications / Unsuitability

Wearing the product is only recommended for the following clinical conditions following consultation with your doctor:

- skin diseases/injuries in the affected part of the body, especially inflammatory conditions such as swelling, redness and overheating
- sensory disturbances and movement disorders, e.g. with diabetes

- in case of infants or persons who cannot notice if the product is placed uncomfortably
- lymph drainage disorders or unclear soft tissue swellings even outside the bandage area
- open wounds or internal injuries
- allergic reactions to the bandage

Instructions for safe use of the product

- There are no known side-effects that affect the whole body. Proper use is a prerequisite.
- Product liability shall be excluded in case of improper use.
- Immediately remove the bandage if it is too tight or if you find it uncomfortable!
- Use by the hour in case of home users.
- If your symptoms worsen or you experience any unusual changes, tell your doctor immediately.
- Any major incident involving the product should be reported to the manufacturer and the relevant authority in the Member State where the user and/or patient is located.

For Germany

Federal Institute for Drugs and Medical Devices
Medical Devices Division
Kurt-Georg-Kiesinger-Allee 3
53175 Bonn
<https://www2.bfarm.de/medprod/mpsv/>

For Austria

Federal Ministry of Health
Dept. Pharmaceuticals and Medical Devices
Radetzkystrasse 2, 1030 Vienna
Email: meddev@bmgf.gv.at

For Switzerland

Swissmedic, Swiss Agency for
Therapeutic Products
Medical Devices Division
Hallerstrasse 7, 3012 Bern, Switzerland

Do not use

While sleeping.

The comfortable Velcro fastener fixes the joint bandage by simply putting it on. There is no defect if small fibres are pulled out of the bandage's surface material when the Velcro fastener is opened.

Use / applying the bandage


Fig. 1: Place the bandage level above the knee. The non-slip dimples are positioned on the side facing the skin, the first Velcro fastener on the side facing away from the skin. In order to stabilise the outer side of the knee, apply the bandage to the inner side of the leg (see sketches). In order to stabilise the inner side of the knee, apply the bandage the other way. Wrap the bandage around the leg while applying slight tension and attach it to the first Velcro fastener.

Fig. 2: Then guide the bandage over the knee joint downwards in a diagonal motion. The stabilisation can be adjusted through varying degrees of tension. Ensure a crease-free fit when wrapping.

Fig. 3: Finally, wrap the bandage around the leg below the knee joint and fix the Velcro fastener at any point.

Cleaning

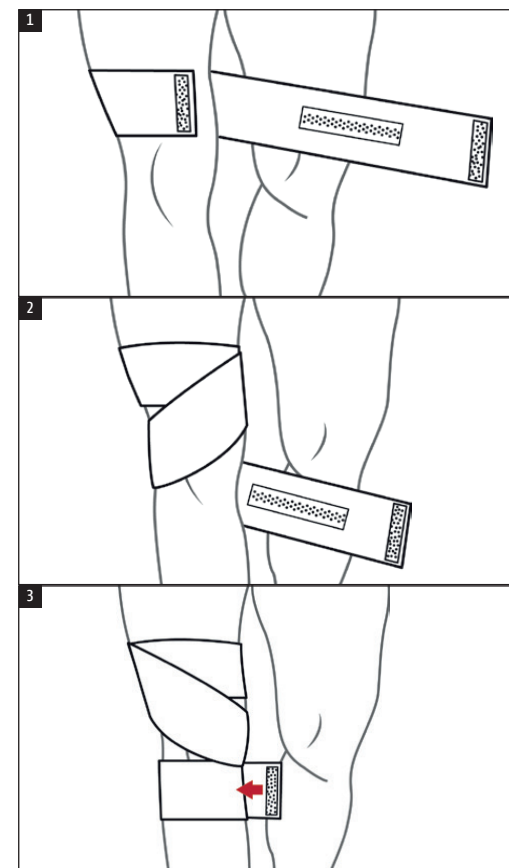It can be washed with the household laundry – see the care instructions. Close the Velcro fasteners before washing to prevent damage to other items of laundry.

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Biber Umweltprodukte Versand GmbH
Hydas GmbH & Co. KG, Hirzenhainer Straße 3,